


EXAM ROOM

4 Steps to Disinfection

1 Pre-clean by removing any visible soils

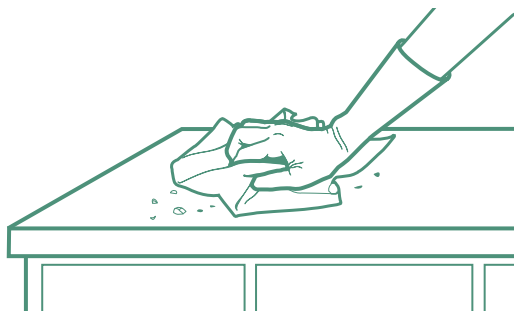


2 Apply the Rescue™ disinfectant using product of choice¹



CONCENTRATE DILUTION CHART

AREA	DILUTION	CONTACT TIME ¹	DESCRIPTION ¹
Non-Animal Related Areas (Hallways & Offices)	1:128 (1 oz/gal)	3 Minutes	Non-Food Contact Sanitization
Daily Disinfection of all Animal Housing Areas and Enclosures	1:64 (2 oz/gal)	5 Minutes	Virucide, except for *Canine Parvovirus
		10 Minutes	Bactericide
High Risk/Quarantine Areas	1:32 (4 oz/gal)	10 Minutes	Virucide (including Canine Parvovirus)
Confirmed Disease/Outbreak Conditions	1:16 (8 oz/gal)	5 Minutes	Virucide (including Canine Parvovirus), Bactericide, and Fungicide

3 Ensure surface remains wet for the required contact time¹

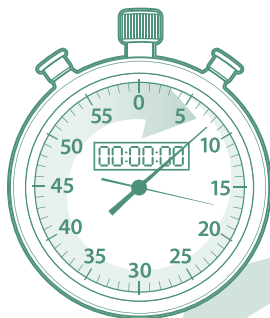

4 Allow surface to air dry or wipe dry²



PRODUCT OPTIONS:

Rescue™ Concentrate¹

- Refer to dilution chart

Rescue™ Ready to Use¹

- 1 minute contact time

Rescue™ Wipes¹

- 1 minute contact time

SURFACES CHECKLIST:

- Exam Room table
- Stethoscope
- Otoscope
- Blood Pressure Equipment
- Door Handle
- Computer Equipment
- Sink
- Thermometer
- Nail Clippers
- Drawer Handles
- Floors
- Scale

For more information visit:
ViroxAnimalHealth.com

¹ Refer to product label for use instructions and/or reference sheet for complete list of claims and use instructions

² Some surfaces may require rinsing

Brought to you by:

