

PREvention™ HLD



8 Minute **H**igh **L**evel **D**isinfectant for Instruments & Devices *Powered by* **AHP**[®] Patented

Prevention™ HLD8 is the **only** high level disinfectant (HLD) that doesn't compromise safety, productivity, or sustainability for strength. Powered by Accelerated Hydrogen Peroxide® (AHP®), the benefits of Prevention™ HLD8 include:

- *Rapid 8-minute contact time*
- *Up to 21 day re-use*
- *Ready to use (no mixing or activation required)*
- *No special ventilation required*
- *Leaves behind no toxic residues*
- *Biodegradable ingredients*

Why Use a HLD?

Instruments that come into contact with mucous membranes or non-intact skin increase the risk of transmission of infectious pathogens and require a higher level of care when reprocessing.

For Use On:

- *Nebulizers*
- *Endotracheal Tubes*
- *Endoscopes*
- *Ultrasound Probes*
- *Otoscope Cones*
- *Speculums*
- *Other semi-critical devices*

PREvention™ HLD8 is the **Deliberately Different™** solution for your high level disinfection needs!

OSHA COMPLIANCE

- An ideal HLD will be **non-toxic**, require **no special ventilation**, and will not induce dermatitis or respiratory distress

PRODUCTIVITY

- An ideal HLD will have a **fast contact time** and will have a long **re-use period**
- An ideal HLD will only require a **single rinse**, whereas some chemistries require up to 5 rinses

SUSTAINABILITY

- An ideal HLD will not require **detoxification** and will have **biodegradable** properties allowing it to be poured safely down the drain
- An ideal HLD will not leave behind **toxic residues** that are harmful to patients, staff, and the planet

PRODUCT	RATING
2% Accelerated Hydrogen Peroxide® (AHP®)	★★★★★
H ₂ O ₂ ≥6%	★★★☆☆
Peracetic Acid	★★★☆☆
Glutaraldehyde	★☆☆☆☆
OPA	★☆☆☆☆

2% Accelerated Hydrogen Peroxide® (AHP®)	★★★★★
Peracetic Acid	★★★★☆
OPA	★★★★☆
H ₂ O ₂ ≥6%	★★★☆☆
Glutaraldehyde	★☆☆☆☆

2% Accelerated Hydrogen Peroxide® (AHP®)	★★★★★
H ₂ O ₂ ≥6%	★★★★★
Peracetic Acid	★★★★★
OPA	★★★☆☆
Glutaraldehyde	★☆☆☆☆


How to Use PREvention™ HLD8

Prevention™ HLD8 is a high level disinfectant powered by Accelerated Hydrogen Peroxide® (AHP®), a **FASTER, RESPONSIBLE, and SUSTAINABLE** disinfectant technology. Prevention™ HLD8 quickly provides high level disinfection without the use of harmful chemicals or need for special ventilation systems.

How to Use^β

1 POUR


Pour fresh solution[‡] of Prevention™ HLD8 into clean soaking tray, no activation or dilution is required

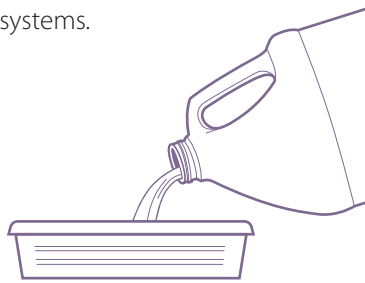

2 CLEAN

A) Clean all instruments using a detergent*

B) Thoroughly rinse and dry instruments

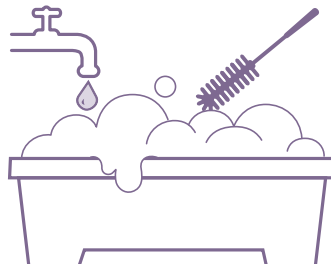


3 DISINFECT

A) Fully immerse pre-cleaned instruments in Prevention™ HLD8 solution

B) Wait **8 minutes**

C) Rinse instrument with potable or sterile water and dry[†]


D) Store in clean, closed container

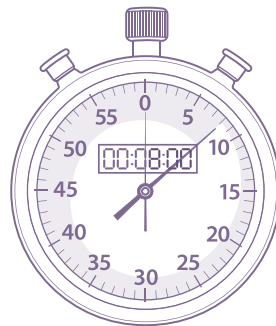

How to Validate


Use Prevention™ HLD8 Test Strips to validate the required concentration of hydrogen peroxide during the re-use period up to 21 days

Dip
Hold
Match

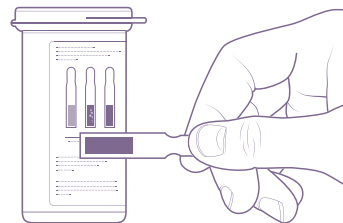

When to Discard Solution^{**}

- Up to 21 days
- If the solution is visibly soiled
- If indicator strip fails test

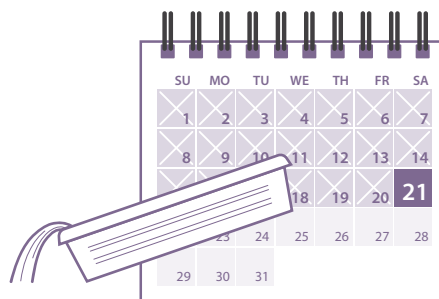

^β Follow product label directions for use

[‡] Solution to be used at room temperature

* Cleaning can be done either manually or mechanically.

** For disposal of Prevention HLD8 follow local and state regulations.

[†] Refer to device manufacturer's recommendations for rinsing and drying instructions.

HLD

POWERED BY
AHP[®]
Patented

Product Order Information

PREVENTION™ HLD8 HIGH LEVEL DISINFECTANT

- FDA Listing No.: D295084
- Format Size: 1 Gallon
- Product Code: 25505
- Packaging Specifications: 4x1 Gallon

PREVENTION™ HLD8 HIGH LEVEL DISINFECTANT TEST STRIPS

- FDA Listing No.: D295198
- Format Size: 60 Test Strips
- Product Code: AHPHLD8
- Packaging Specifications: 1x60 Test Strips

Prevention™ HLD8 High Level Disinfectant Test Strips were designed to validate the Prevention™ HLD8 solution for up to 21 days. Test Strips are available for purchase through distribution.

For more information, visit:

ViroxAnimalHealth.com

Virox Animal Health

2770 Coventry Road
Oakville, ON L6H 6R1 CANADA
Toll Free: 1-800-387-7578

Deliberately.
Different. 

AHP[®]
Patented

ACCELERATED[®]
HYDROGEN PEROXIDE
Patented

VIROX[™]
ANIMAL HEALTH